

Shining® Scan Checklist for Scanning (Removable Dentures - Bite Rim)

Make sure **Intraoral Mode** is enabled

Make sure **Edentulous Mode** is enabled when scanning edentulous arches

Scan the working jaw as a full arch, capturing all the landmarks required for a denture case

Scan the antagonist as a full arch, capturing all the landmarks required for a denture case

Turn **off the Intraoral Mode**

Scan the bite rim, starting with the intaglio surface of the working jaw (capture in full)

- Continue scanning over the vestibular ridges to capture all sides of the bite rim
-

Seat the bite rim and scan the right-side bite

Keeping the bite rim seated, scan the left-side bite

Add an anterior bite if necessary