

Shining[®] Scan Checklist for Scanning (Removable Dentures - Existing Denture)

Make sure **Intraoral Mode** is enabled

Make sure **Edentulous Mode** is enabled when scanning edentulous arches

Scan the working jaw as a full arch, capturing all the landmarks required for a denture case

Scan the antagonist as a full arch, capturing all the landmarks required for a denture case

Perform a reline/wash impression with existing denture to ensure a good fit

Trim excessive material away from the denture teeth for alignment and design clarity

Turn **off the intraoral mode**

Perform a 360° scan of the existing denture, capturing all teeth and intaglio surfaces

Seat the denture and scan the right-side bite

Keeping the denture seated, scan the left-side bite

Add an anterior bite if necessary

Verify occlusal hits using heat-map