

## Shining® Scan Checklist for Scanning (Implants)

---

Make sure **Intraoral Mode** is enabled

Scan the working jaw soft tissue as a full arch

Punch out the implant site

Seat the scan body on the implant

Take a PA to verify a proper seat

Scan the scan body in full

Remove the scan body and replace the healing cap/tissue former

Scan the antagonist as a full arch

Scan the right-side bite

Scan the left-side bite

Add an anterior bite if necessary

\*\*\*Verify occlusal hits using heat-map \*\*\*